


How to fit DEPEND® Guards for Men

STEP A

Ensure groin & buttocks area is thoroughly cleaned and dried prior to changing DEPEND® Guards for Men

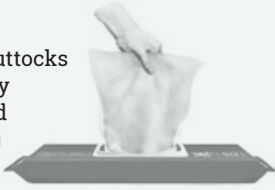


STEP B

Identify the inside and outside of the product.

- The inside (side placed against the skin) has a blue absorbent layer and leakage barriers.
- The outside (side placed against the underwear) has a paper strip covering an adhesive area for attaching the guard to underwear.

Identify the ifront and back of the product.

- The back of the guard cups the testicles.
- The arrows on the paper strip on the outside of the guard point to the front of the guard.

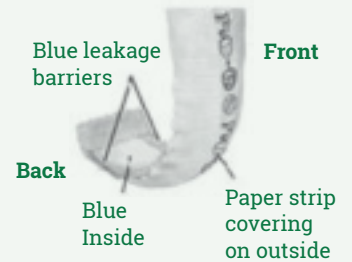

STEP C

Unfold the guard. Stand the leakage barrier shields into an upright position to help form a barrier.



FITTING TO UNDERWEAR



1. Peel the paper strip off the expose the adhesive strip on the outside of the guard.



2. Place guard inside snug-fitting underwear (not boxer shorts).



3. Secure guard by gently pressing underwear fabric against adhesive strip.



4. Make sure underwear is fitted snugly to the body.

FITTING TO BODY



1. Place the guard against the body so the back side cups the testicles. Be sure that the penis is pointing down the centre of the guard.



2. Peel the paper strip off to expose the adhesive strip on the outside of the guard, away from the skin.



3. While holding the guard in place, pull firm-fitting underwear over the guard and into place.



4. Secure guard by gently pressing underwear fabric against adhesive strip. Make sure underwear is pulled up snugly.

Tips

- Make sure wearer has snug-fitting underwear (not boxer shorts).
- Dispose of soiled guards in a sanitary manner. Do not flush.
- Wash hands after handling soiled guards.
- Do not get powder, lotion or oil on adhesive strips. These substances may cause the strips not to stick properly.



For more information or a free sample call
1800 028 334 or visit www.dependcare.com.au